



Rondish SafeTime Alert Monitoring System

NEW!


Introducing the most advanced Bed/ChairWatcher™ monitor of all...

SafeTime Alert (BSA-01) is a universal monitor which will work with any pad but it has a unique feature: it is designed to work with high-quality SilverLining SafeTime pressure sensor pads to give the very special benefit that it alerts the caregiver when the pad expires, whether after 90 days or 1 year.

The SafeTime concept means no more overuse of worn and dilapidated pads, prone to faults and hygiene problems, which have been kept beyond their useful and safe life. This acts as a reminder "the used by" date has expired so the safety of the patient is not compromised.

You can better insure the safety of patients and protect the reputation of your facility, by minimizing the chance of caregivers inadvertently using an old, non-functioning pad, and, as a result, missing wandering and falls by patients in their care.

With the highest warranty of effective operation, not only do you give optimum patient care and caregiver service, but also minimize the likelihood of litigation due to equipment malfunction.

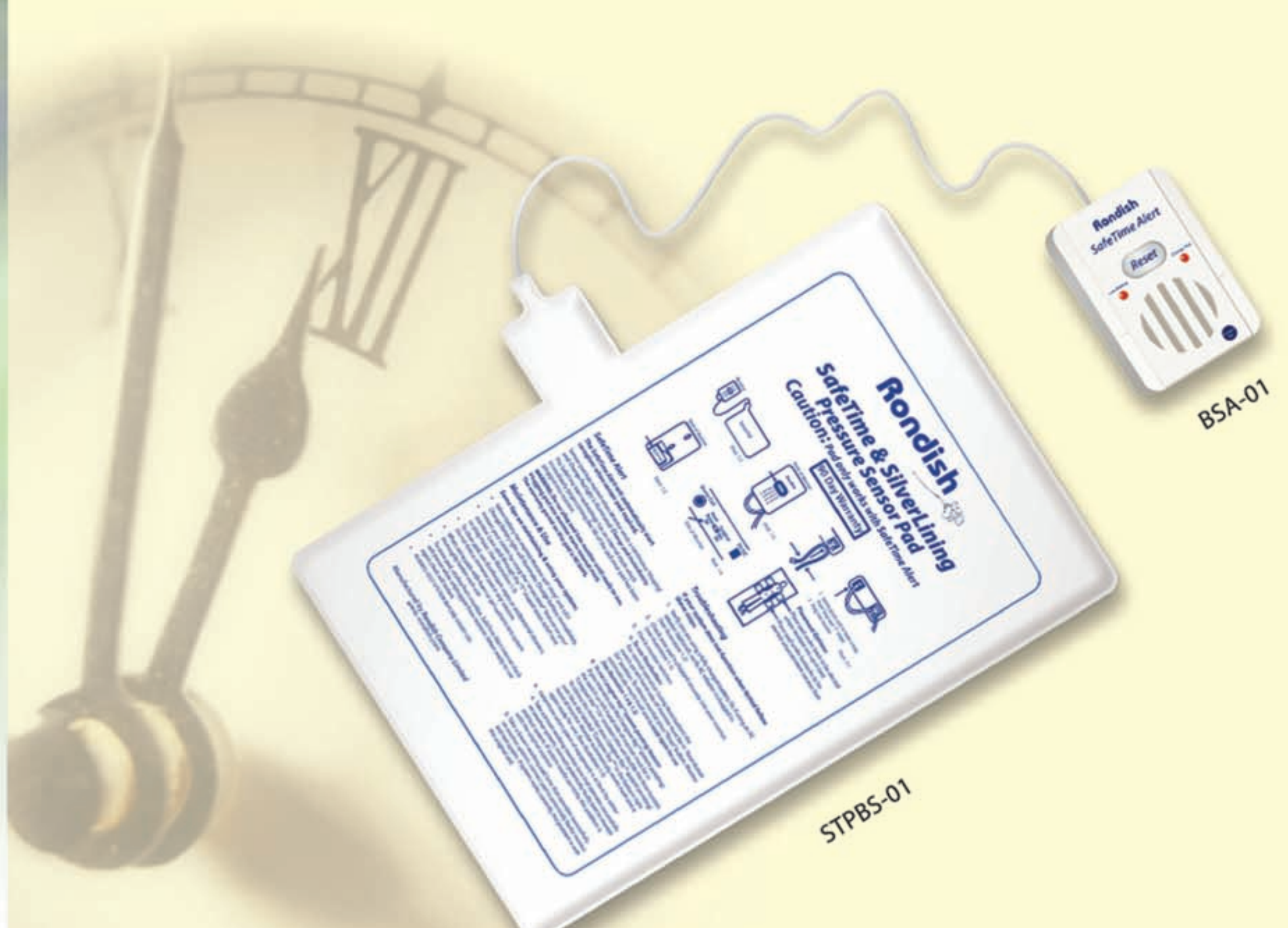

SafeTime Alert Monitoring System

- Easy-to-use Rondish Bed/ChairWatcher monitors let caregivers know when fall-prone patients get out of beds or chairs and start to wander unsupervised.
- The new SafeTime Alert works with all pads and floor mats but its unique benefit is that it works with special SafeTime pressure pads equipped with a timer micro-chip to provide continuous timing of the safe life of the pad (90 days or 1 year).
- As soon as the SafeTime sensor pad has expired, the monitor emits a continuous "chirping" and the "Change Pad" light is illuminated. It's telling you it's time to change the pad!
- The SafeTime Alert technology has an internal clock which immediately starts timing the life of the pad automatically, as soon as the pad is plugged into the monitor. Plug in the pad only when ready to use, since this starts the internal clock.
- It's a safety auto-reset system which auto resets whether the button is pressed or not. i.e. can't be switched off – always ready for action!
- Available with Caregiver Key (CK-01), to avoid patient resetting the system, or using standard reset (switchable by caregiver).
- Many other extras are available, such as bed and chair painted metal brackets, wall brackets, protective soft plastic covers which are anti-slip for table top use, CE and UL approved AC adaptors, and a Nursecall cable (NC-01) with standard plugs for integration with existing wired Nursecall systems.



Specifications

- Battery life for monitor: up to 9 months
- Pad life: 90 days or 1 year (other client-specified times available for large special orders)
- CE & UL versions for Europe and US
- Patent applied for
- Dimensions 12 x 7.7 x 2.5 cm plastic case
- Blue LED alarm indicator light
- Adjustable sounder
- Sound output with beeps to indicate bed number - 3 options
- Volt free normally open and normally closed alarm outputs
- Telephone type socket (miniature) for bed pad with strain relief
- Low battery detection and indication
- Missing AC/DC supply detection and indication
- 9 Volt DC operation using either a PP9 battery or 9V adaptor power unit
- Floor mat sensor operation
- Many mounting options
- Simple operation
- Low maintenance

Distributed By:



Hong Kong Office:
Rondish Company Limited
Tel: +852 2545 2057
Fax: +852 2541 7411
info@rondish.com
www.rondish.com



USA Office:
RondishUSA LLC
Tel: +1 858 729 0251
Fax: +1 815 346 5396
sales@rondishUSA.com
www.rondishUSA.com